




Results are important for both surgeons and patients

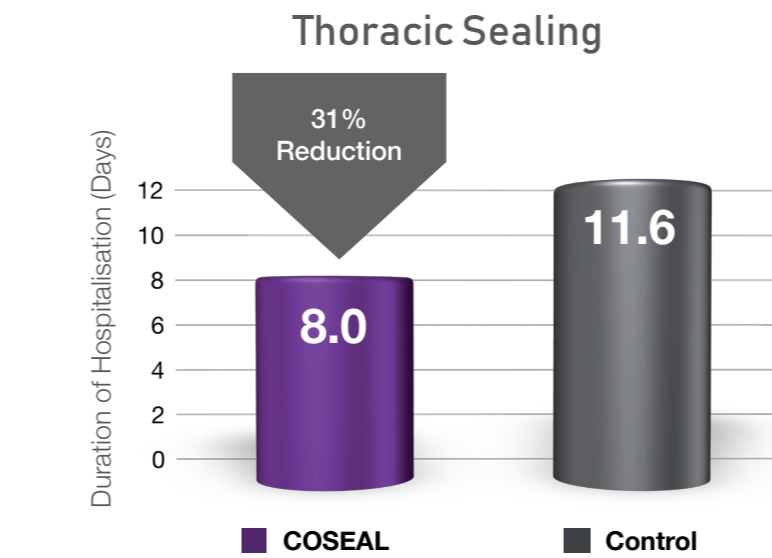

COSEAL Surgical Sealant significantly reduced the length of hospitalisation for patient undergoing lung surgery compared to controls ($p = 0.009$)³ Adapted from Venuta et al.

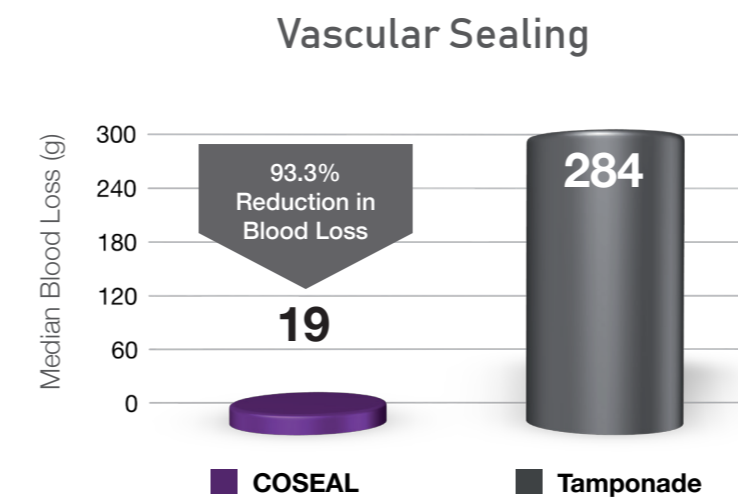

In this preclinical study blood loss was significantly reduced in the COSEAL group ($19 \pm 31g$, Median = 1.7g) compared with the tamponade control group ($284 \pm 151g$, Median = 232g) ($p = 0.05$)² Adapted from Hill et al.

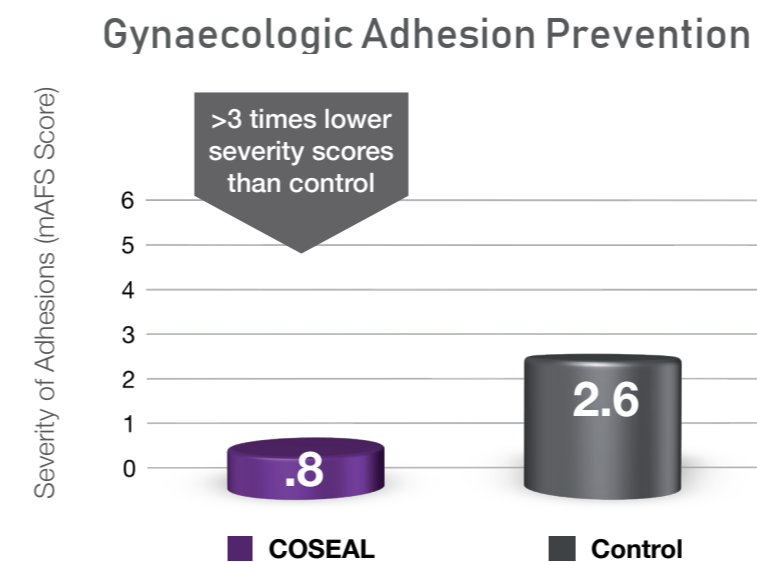


The Coseal group had more than 3 times lower severity scores than control ($p = 0.01$) at the second look procedure⁴. Adapted from Mettler et al.

Biocompatible

COSEAL Surgical Sealant is largely composed of water (~80%) while approximately 20% is comprised of two different polyethylene glycol (PEG) molecules and a HCl buffer. PEG is a biologically inert material that is commonly used in surgical devices.


Action

COSEAL consists of two different PEG molecules that cross link with each other and form a covalent bond with proteins from underlying tissue to form an absorbable, adherent and biocompatible hydrogel layer.

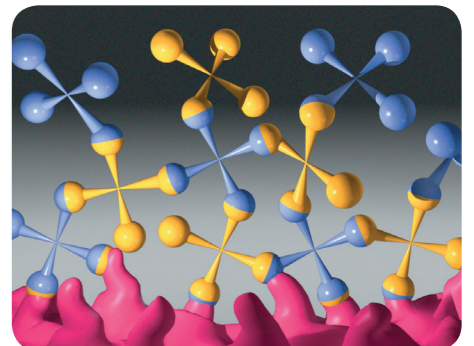

The PEG layer is impermeable to the fibrin rich exudate which is present during postoperative healing, preventing the formation of tenacious bands of adhesiogenic tissue. The rapid polymerisation of COSEAL completes within 60 seconds to form a continuous layer of burst resistant, elastic, low friction hydrogel.

Products

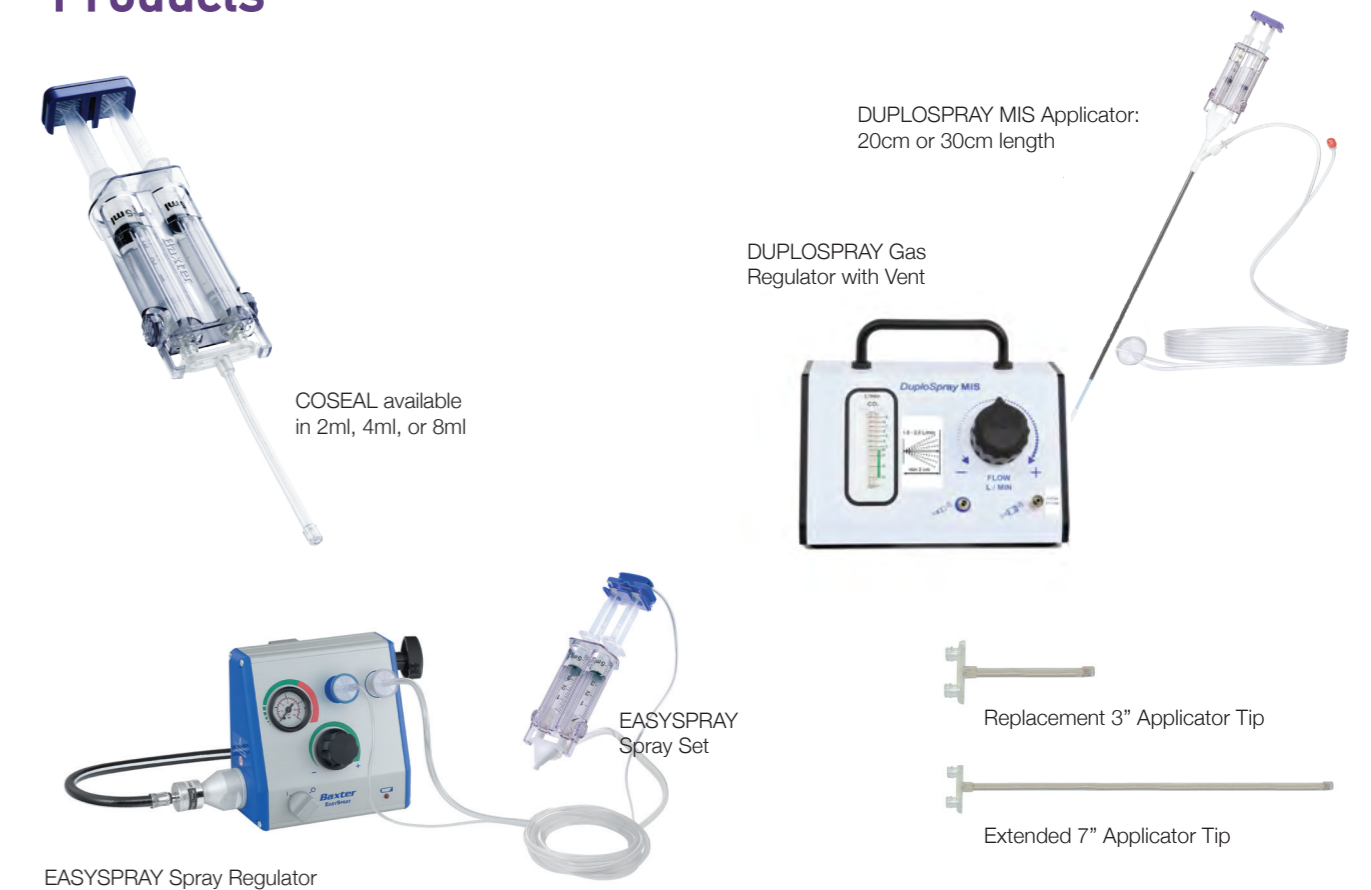

Control and Protect

Synthetic Sealing In Cardiac, Vascular and Thoracic Surgery Adhesion Prevention In Cardiac and Gynaecologic Surgery

Adherent to tissue

COSEAL Surgical Sealant forms a mechanical lock to synthetic materials and covalently bonds with the proteins in the underlying target tissue⁵

Conformable & elastic

COSEAL conforms to the anatomy of underlying target tissue, is elastic and non-restrictive²

Site specific

The components of COSEAL rapidly polymerise within ~5 seconds and the resulting hydrogel attains full strength within 60 seconds (sealant to be left undisturbed during this time)²

Controlled delivery


With a choice of 5 different applicators, COSEAL can be applied and delivered how and where you need it

Simple to use

COSEAL can be prepared within 2 minutes and it is stored at room temperature (below 25 °C / 77 °F)⁵

Synthetic hydrogel technology

Two distinct synthetic polyethylene glycol (PEG) polymers crosslink together to form COSEAL, which is hydrolysed over a period of 7-30 days (according to preclinical studies)*^{2,5}

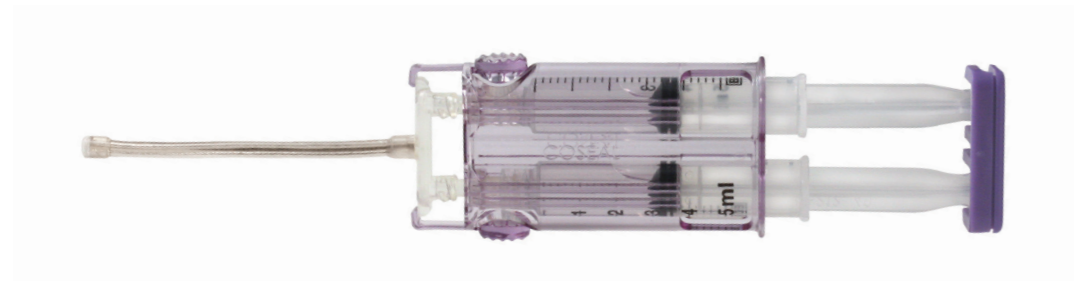

* Preclinical studies suggested resorption in 7-30 days. The rate of resorption can differ as it is dependent on several factors, including the amount of product applied in situ and the site of use.



Coseal
Surgical Sealant

Product Code	Product Information
934073	2ml COSEAL Surgical Sealant
934074	4ml COSEAL Surgical Sealant
934075	8ml COSEAL Surgical Sealant
934033	Replacement 3" Applicator Tip
934034	Extended 7" Applicator Tip
1504275	EASYSpray Spray Regulator
0600021	EASYSpray Spray Set
0600043	DUPLOSPRAY MIS Applicator 20cm
0600044	DUPLOSPRAY MIS Applicator 30cm
0600123	DUPLOSPRAY MIS Regulator



COSEAL Surgical Sealant Device Summary Please refer to your local IFU for further information.

Indications:

- Sealing suture lines along arterial and venous reconstructions.
- Enforcement of suture and staple lines in lung resection procedures.
- Patients undergoing cardiac surgery to prevent or reduce the incidence, severity and extent of post surgical adhesion formation.
- Patients undergoing laparotomic or laparoscopic gynecological surgery as an adjunct to good surgical technique intended to reduce the incidence, severity and extent of post surgical adhesion formation.

Composition: 1) Two synthetic polyethylene glycols. 2) Dilute hydrogen chloride solution. 3) Sodium phosphate / sodium carbonate solution.

Contraindications: Do not use COSEAL as sealant for bronchial stump, sleeve resection, decorticated lung or where pleural adhesions are desired.

Warnings: Do not inject into blood vessels. Do not use in place of sutures, staples or mechanical closure. Increases in volume by four times within 24 hours of application. Additional swelling occurs during resorption. A thin layer of product (1ml/10cm²) is recommended in compression sensitive areas. Do not use in contaminated pulmonary resection. Not for use as an adherent.

Disclaimer: Refer to IFU.

References

1. Wallace D G et al. A Tissue Sealant Based on Reactive Multifunctional Polyethylene Glycol. J Biomed Mater Res (Appl Biomater) 2001; 58: 545-555
2. Hill A et al. Treatment of Suture Line Bleeding with a Novel Synthetic Surgical Sealant in a Canine Iliac PTFE Graft Model. J Biomed Mater Res (Appl Biomater) 2001; 58: 308-312
3. Venuta F et al. Use of a polymeric sealant to reduce air leaks after lobectomy. J Thorac Cardiovasc Surg 2006; 132(2): 422-423
4. Mettler L et al. A safety and efficacy study of a resorbable hydrogel for reduction of postoperative adhesions following myomectomy. Human Reproduction 2008; Vol.23(5): 1093-1100
5. Baxter Healthcare SA 2021. COSEAL Instructions for Use.

CE 0123

Advanced Surgery - Advancing the Art of Healing

Baxter, COSEAL, Easyspray and Duplospray are trademarks for Baxter International Inc., its subsidiaries or affiliates.

NOR-AS13-230005 July 2023

Baxter

www.baxter.se
Baxter Medical AB
P 08 632 64 00

www.baxter.dk
Baxter A/S
P 4816 6400

www.baxter.no
Baxter AS
P 22 58 48 00

www.baxter.fi
Baxter Oy
P (09) 8621 111



Coseal
Surgical Sealant



5 Specialties,
3 Needs in Theatre,
1 Solution for All

Protecting what matters is a clear choice

Baxter